## Oxygen Calibration

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To perform the oxygen calibration, click on «Calibration» in the Home Screen, then click «Oxygen Calibration». Choose between a one-point calibration and two-point calibration and follow the instructions on the screen. When carrying out the two-point oxygen calibration, ensure you use 100% oxygen. For further information please refer to the user manual.

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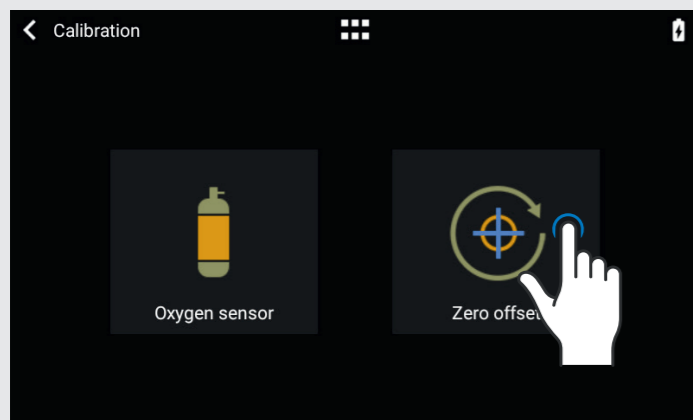
## Zero Offset Calibration

### Null Offset Kalibrierung

To perform a zero offset calibration make sure all flow and pressure connections are open. Please click on «Calibration» in the Home Screen, then «Zero Offset» and follow the instructions on the screen. For further information consult the user manual.

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## User Manual

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The user manual for the CITREX H5 can be found on our website at the link below. Please read the operating instructions carefully and use the product as recommended.

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! Read the user manual carefully before using the CITREX H5.  
Online at [www.imtanalytics.com/analyser\\_manuals](http://www.imtanalytics.com/analyser_manuals)

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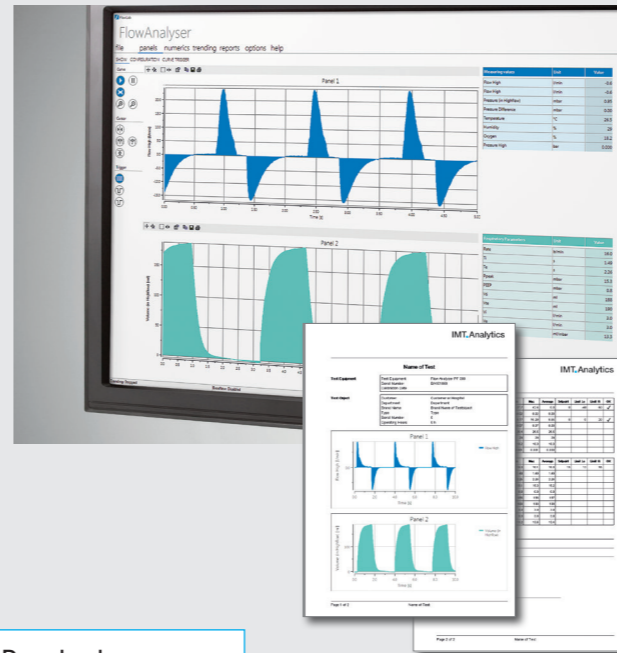
## FlowLab Software

### FlowLab Software

View your flow, pressure and volume measurements comfortably on your computer screen while simultaneously saving respiratory parameters. The FlowLab License is activated on your CITREX H5.

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! FlowLab Software Download:  
[www.imtanalytics.com/FlowLab](http://www.imtanalytics.com/FlowLab)

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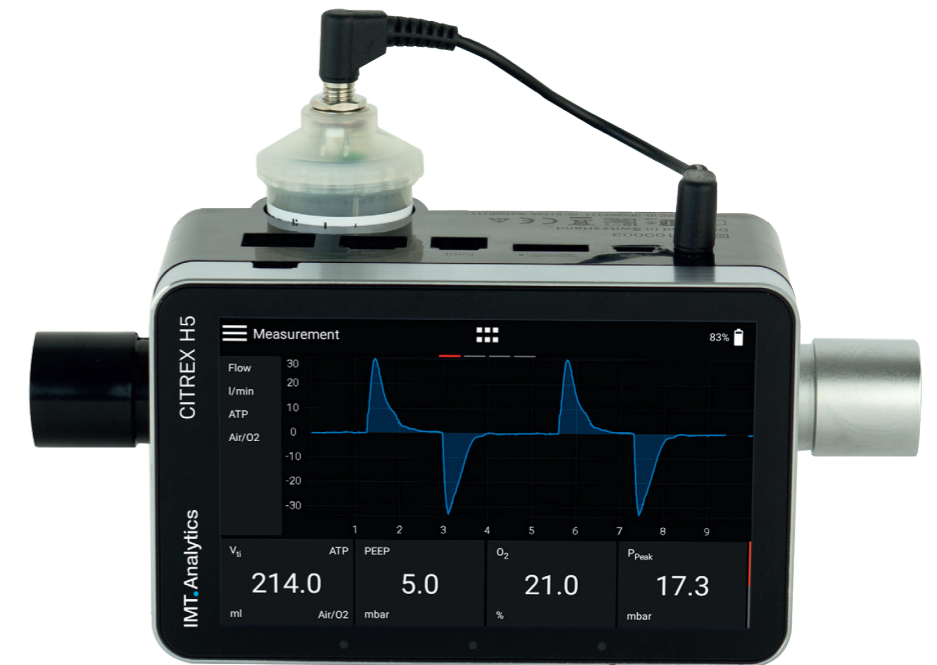
! To guarantee the accuracy of your measurements, your CITREX must be recalibrated by IMT Analytics yearly.

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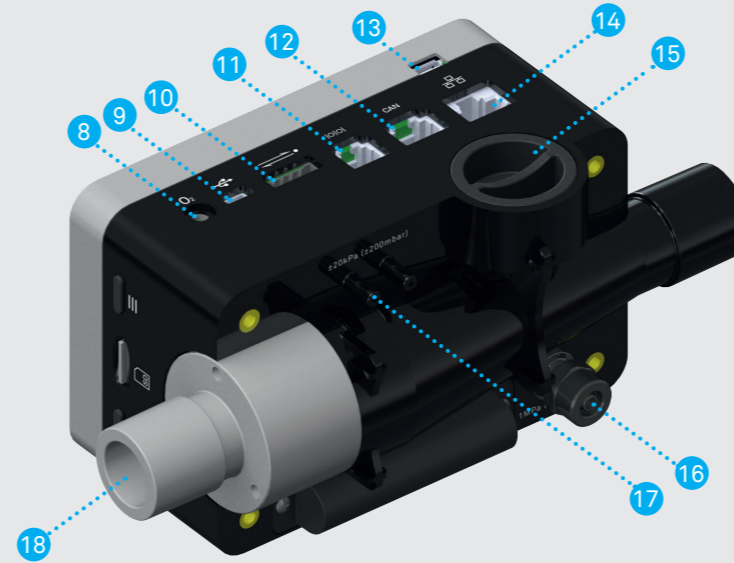
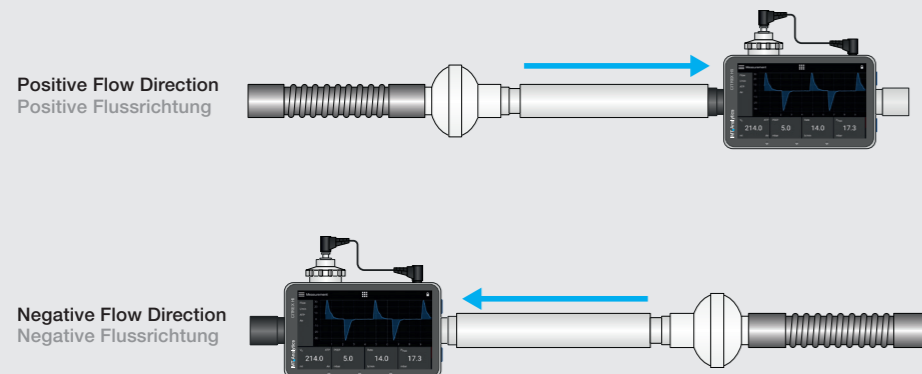
We recommend using a protection filter and the Laminar Flow Tube for absolute accuracy. The filter protects the device from contamination and dust. Please make sure the filter is placed right before the laminar flow tube.

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The measured gas must be free of oil, grease and dust.

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The device setup can affect the accuracy of the flow measurement. Tight bends, kinks or dents should be avoided. For more information please consult the user manual.

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The CITREX H5 is a powerful device for ventilator calibration and verification. We recommend you follow the measurement setup below.

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Measurement must be performed **with the filter**. The measured gas must be **free of oil, grease and dust**.

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